Gutta Percha Points

2021. 10. 06. DM-119-4

SYMBOLS

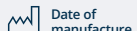
nl Fabrikant
fr Fabricant
de Hersteller
it Produttore
pt Fabricante
es Fabricante
tr Üretici

nl Partijcode
fr Code de lot
de Chargennummer
it Codice Lotto
pt Código de lote
es Código de lote
tr Parti Kodu

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

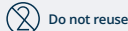
CE - Mark and identification number of notified body.
1639 Product conforms to the essential requirements of the Medical Device Directive 93/42/EEC

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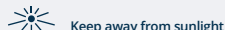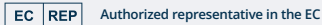Manufacturer
META BIOMED CO., LTD.
270, Osongsangmyeong 1-ro,
Osong-eup, Heungdeok-gu, Cheongju-si,
Chungcheongbuk-do, Republic of Korea

Tel +82-43-216-0433
Fax +82-43-217-1988
E-Mail info@meta-biomed.com
www.meta-biomed.com

Meta Biomed Europe GmbH
Wienstr. 35, 45473 Mülheim an der Ruhr,
Germany

Tel +49-208-30991910
Fax +49-208-30991999

en INSTRUCTION FOR USE

Gutta percha points are root canal obturation material. It is used to fill the cleaned, shaped and irrigated root canal space.

- Time saving with easy filling
- For faster and easier root canal obturation
- Variable tapers & sizes
- Length marking available
- Excellent radiopacity
- Long shelf life and freshness assured
- Suitable for vertical and lateral condensation

INDICATION AND PURPOSE

- 1 Gutta percha point is an integral material in promoting periapical healing and preventing disease progression.
- 2 Endodontic therapy system using the gutta percha point obturated impermeable to liquid and bacteria

INSTRUCTION FOR USE

- 1 Prepare root canal by file.
- 2 After clean and dry the canal with EDTA solution, apply the sealer into the root canal wall by paper points.
- 3 Select master gutta percha point matched in prepared root canal.
- 4 Irrigate a gutta percha point with 5.25% NaOCl disinfecting solution.
- 5 Insert mater points in to the root canal as working length.
- 6 Insert accessory points into the root by finger spreader.
- 7 Confirm a filling by the x-ray.

CAUTIONS

- 1 Do not use this product for any patients with a history of hypersensitivity reactions to it, such as a rash, dermatitis, etc.
- 2 Some patients have an allergic reaction to Trans 1,4 Polysoprene materials and/or even experience hypersensitivity. Do not use this product for them
- 3 This product is intended only for use for the tissue inside the mouth. Be careful to prevent it from coming into contact with eyes. If material has come into contact with eyes or skin, wash immediately with plenty of water and then consult a doctor.
- 4 Non-specialists are prohibited from using this product. Do not use this product for any purposes other than its intended purposes.
- 5 Do not prescribe an internal medicine
- 6 Should be careful to avoid overfilling or dead space
- 7 Be sure to take measures for the protection of the respiratory tract before
- 8 Always close the slide lid after use space
- 9 Do not reuse. Reusing these products increases the risk of cross-contamination and/or breakage.

STORAGE

Store at room temperature of 1°C–30°C (33°F–86°F)

SHELF LIFE

4 years from manufacture date

Available Size

Type	Sizes
Standardized (ISO Specifications)	10, 15, 20, 25, 30, 35, 40, 45, 50, 55, 60, 70, 80, 90, 100, 110, 120, 130, 140
Non-Standardized (Accessory Sizes)	X-Fine, Fine-Fine, Medium-Fine, Fine, Fine-Medium, Medium, Medium-Large, Large, X-Large, F1/F2/F3
Special Taper (.04, .06, .08)	15, 20, 25, 30, 35, 40, 45, 50, 55, 60, 70, 80
Japanese type	JF-S, JF-M, JF-L, JS, JM, JL, JL-S, JL-M, JL-L - De Trey Type : XX-F, X-F, Fine, Medium

COMPONENTS

- Available packaging

- 1 6 compartment dispenser - Code: P.21/ Regular packing points: 120 points
- 2 Sliding type dispenser - Code: P.71/ Regular packing points: 120 points
- 3 6 Val packing (Square) - Code: P.31/ Regular packing points: 120 points
- 4 6 Val packing (Round) - Code: P.31/ Regular packing points: 120 points
- 5 4 compartment dispenser - Code: P.51/ Regular packing points: 100 points
- 6 Endo Organizer (Spill Proof 12 Compartments) - Code: P.11/ Regular packing points: 240points/ Size: ISO 15-80

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