# Meta Etchant

37% Phosphoric Acid Etching Gel

 META BIOMED

2020.10.15. DM-57-14

 **Manufacturer**

**META BIOMED CO., LTD.**

270, Osongsaeangmyeong 1-ro,  
Osong-eup, Heungdeok-gu, Cheongju-si,  
Chungcheongbuk-do, Republic of Korea

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**CE - Mark and identification number of notified body.**  
Product conforms to the essential requirements of the Medical Device Directive 93/42/EEC

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EC REP Authorized representative in the EC

nl Erkende vertegenwoordiger in de EG  
fr Représentant autorisé dans l'UE  
de Bevollmächtigter/in in der EU  
it Rappresentante autorizzato nella CE  
pt Representante autorizado na CE  
es Representante autorizado en la CE  
tr AB'de yetkili temsilci

en **INSTRUCTION FOR USE**

**37% Phosphoric Acid Etching Gel**

Meta Etchant is gel for enamel and dentin conditioning for sealing and direct composite restorations, adhesive cementation of inlays, crowns, bridges veneers, adhesive bridges and bonding of orthodontic attachments.

### Instructions

- ① Isolation: Rubber dam is the preferred method for isolation.
- ② Cavity preparation: Prepare cavity with minimal tooth reduction.
- ③ Tooth conditioning  
Dentin pretreatment: Apply 37% Meta Etchant gel to available enamel/dentin for 15 seconds.
- ④ Rinse and Blot dry: Remove the gel by suction using aspirator tube and rinse for 20 seconds. Blot dry conditioned area with a moist cotton pellet.

### Side effects

None known to date

### Warning

Meta Etchant is corrosive and contains Phosphoric acid. Protective eyewear is recommended when using Meta Etchant. Avoid contact with eye and skin. If accidental eye contact occurs, flush immediately with water and consult a physician.

### Cautions

- For dental use only.
- Keep away from children.
- Do not use Meta Etchant after date of expiration.

### Storage

- Store at 1°C-30°C(33°F-86°F)
- Refrigeration is recommended to extend shelf life

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