NETC

Non-Eugenol Temporary Cement

 META BIOMED

2023. 10. 18. DM-37-19

SYMBOLS

 Manufacturer

nl Fabrikant
fr Fabricant
de Hersteller
it Produttore
pt Fabricante
es Fabricante
tr Üretici

 Batch Code

nl Partijcode
fr Code de lot
de Chargennummer
it Codice Lotto
pt Código de lote
es Código de lote
tr Parti Kodu

 Consult instructions for use

nl Raadpleeg de gebruiksaanwijzing
fr Consulter le CONSIGNES D'UTILISATION
de Gebrauchsanweisung beachten
it Consultare le istruzioni per l'uso
pt Consulte as instruções de uso
es Consulte las instrucciones de uso
tr Kullanım Talimatına Başvurun

 Date of manufacture

nl Productiedatum
fr Date de fabrication
de Herstellungsdatum
it Data di produzione
pt Data de fabrica
es Fecha de caducidad
tr Üretim tarihi

 Use-by date

nl Houdbaarheidsdatum
fr Date limite d'utilisation
de Verwendbar bis
it Data di scadenza
pt Data de validade
es Fecha de caducidad
tr Son kullanım tarihi

 Temperature limit

nl Temperatuurlimiet
fr Limite de température
de Temperaturbegrenzung
it Limite temperatura
pt Limite de temperatura
es Limite de temperatura
tr Sicaklık limiti

 CE
1639

CE - Mark and identification number of notified body.
Product conforms to the essential requirements of the Medical Device Directive 93/42/EEC

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 Authorized representative in the European Community

nl Gemachtigde vertegenwoordiger in de Europese Gemeenschap
fr Représentant autorisé dans la Communauté européenne
de Bevollmächtigter in der Europäischen Gemeinschaft
it Rappresentante autorizzato nella Comunità Europea
pt Representante autorizado na Comunidade Europeia
es Representante autorizado en la Comunidad Europea
tr Avrupa Topluluğundaki Yetkili temsilci

 Manufacturer

META BIOMED CO., LTD.

270, Osongsaeangmyeong 1-ro,
Osong-eup, Heungdeok-gu, Cheongju-si,
Chungcheongbuk-do, Republic of Korea

Tel +82-43-216-0433
Fax +82-43-217-1988
E-Mail info@meta-biomed.com
www.meta-biomed.com

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 INSTRUCTION FOR USE

Non-Eugenol Temporary Cement

NETC Complies with the requirements of ISO 3107 (Type I, Class 2B)

Indication and purpose

The purpose of use of NETC is to wear temporarily on the prosthetic appliance at the dental clinic.

Characteristics

- 1) Easy to mix paste-paste type.
- 2) Especially suitable for eugenol sensitive patients

Instructions for use

1. Preparation before use
 - 1) Check the expiration date of the product and inspect it for foreign materials, impurities or other unusual conditions
 - 2) Follow the directions as documented in the instructions
2. How to use
 - 1) Squeeze equal parts catalyst and base onto a mixing pad.
 - Insert separated tip to dual syringe and squeeze as much as required.
 - 2) Using a spatula thoroughly mix the base and catalyst for 20-30 seconds.
 - 3) Insert paste into temporary crown or bridge and place the prepared dry tooth.
 - 4) Remove excess cement.

Caution

- 1) Do not use this product on a patient with a history of hypersensitivity such as rashes or dermatitis.
- 2) Stop using the product if symptoms of hypersensitivity such as a rash or dermatitis appear.
- 3) The product is only for oral tissues so take care to keep the product away from the eyes.
- 4) If eyes or skin are exposed to un-polymerized product, cleanse with plenty of water and consult a doctor.
- 5) This product is intended only for its primary purpose and should not be used for any other purpose.
- 6) Remove residual cement from the sulcus or the proximal.
- 7) Use equal quantity base and Catalyst pastes.

Storage

- 1) Store at temperature of 18°C-24°C(64°F-75°F)
- 2) After using the NETC, seal it with the cap immediately.
- 3) Do not store near fire.
- 4) Keep out of reach of children.
- 5) Avoid exposure to direct sunlight

Shelf life

Guarantee 2 year shelf life

Components

- 1 Box (Catalyst 20g or 14g, Base 60g or 40g, Mixing pad 1ea)
- Dual Syringe (11g×2ea, Tip x 2ea, Cap x 2ea, Mixing pad 1ea, Spatula 1ea)

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