Nexcomp

Dental Composite Resin

META BIOMED

2023. 10. 18. DM-60-17

SYMBOLS

Manufacturer

nl Fabrikant
fr Fabricant
de Hersteller
it Produttore
pt Fabricante
es Fabricante
tr Üretici

Date of manufacture

nl Productiedatum
fr Date de fabrication
de Herstellungsdatum
it Data di produzione
pt Data de fabrico
es Fecha de fabricación
tr Üretim tarihi

CE - Mark and identification number of notified body. Product conforms to the essential requirements of the Medical Device Directive 93/42/EEC

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Manufacturer

META BIOMED CO., LTD.

270, Osongsangmyeong 1-ro,
Osong-eup, Heungdeok-gu, Cheongju-si,
Chungcheongbuk-do, Republic of Korea

Tel +82-43-216-0433
Fax +82-43-217-1988
E-Mail info@meta-biomed.com
www.meta-biomed.com

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Dental Composite Resin
Anterior and posterior light cured restorative

INDICATION

- Class I, II, III, IV, V restorations
- Core-buildup to replace missing tooth structure
- Diastema closures
- Direct veneers
- Composite and porcelain repairs

CONTRAINDICATIONS

Nexcomp should not be used if patients are known to be allergic to methacrylate resins.

INSTRUCTIONS

- ① Select the desired type and shade of Nexcomp composite prior to isolating and preparing.
- ② Clean the preparation with a slurry of pumice and water. Rinse and dry the preparation.
- ③ Apply 37% Meta Etchant gel to available enamel / dentin for 15 seconds. Remove gel and rinse for 20 seconds. Blot dry to keep moist, Do not rub.
- ④ Apply Bond to the tooth-Apply a double coat of Meta P&Bond by using microbrush. Gently air dry for 5 seconds. Light cure for 10 seconds. If you apply other product to saturate all internal surfaces or bonding agent, apply according to manufacturer's instructions.
- ⑤ Place composite-Place Nexcomp in increments of 2mm or less in anterior and posterior restorations. Cure for 20 seconds in increments of 2mm.
- ⑥ Finish and Polish-After contouring composite, polish surface with finishing and polishing proximal areas.
- ⑦ Occlusion-Check lateral and centric occlusion. Adjust as necessary.
- ⑧ When using dental curing light, it is recommended to use intensity of 1000-1200 mW/cm² and Wavelength of 420-480nm.

SIDE EFFECTS

- None known to date

CAUTIONS

- Avoid contact of Nexcomp with skin/mucous membrane or eyes
- None polymerized Nexcomp may cause light, reversible irritation or lead to a general sensitization against methacrylates.
- Keep away from children.
- For dental use only
- Do not use after date of expiration.
- Exposure to light may lead to premature polymerization

STORAGE

- Store at 18-24°C(64-75°F)
- Close syringe cap immediately after use.
- Avoid from direct sunlight.

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